

DESCRIPTION

Syvek® Patch is a soft, white, sterile non-woven pad of a cellulosic polymer isolated from microalgae (poly-N-acetylglucosamine).

PRECAUTIONS

There are no known contraindications to the use of Syvek® Patch.

Longer compression may be necessary for patients who are hypertensive or who are obese.

Store at controlled room temperature

15°C to 30°C (59°F to 86°F).

Sterilized with radiation.

Do not resterilize and/or reuse.

Single use only.

NOT MADE WITH NATURAL RUBBER LATEX

INDICATIONS

Syvek® Patch is intended for use under the direction of a healthcare professional.

Syvek® Patch promotes the rapid control of bleeding in patients on anticoagulation therapy.

Syvek® Patch is indicated for use in the local management of bleeding wounds such as vascular access sites and percutaneous catheters or tubes, and surgical debridement.

MPT MARINE POLYMER
TECHNOLOGIES

Innovative Medical Solutions

159 Lorum St., Tewksbury, MA 01876-1716

1-888-666-2560

MPS 500-10 rev 08 05/2018

Made in U.S.A.

The logo for Syvek Patch features a stylized graphic of a person's head and shoulders above the brand name. The head is represented by a large circle with three smaller circles stacked vertically inside it, suggesting a face. Below this graphic, the word "Syvek" is written in a large, bold, sans-serif font, with a registered trademark symbol (®) to its upper right. Underneath "Syvek", the word "PATCH" is written in a smaller, all-caps, spaced-out sans-serif font.

**DIRECTIONS
FOR APPLICATION AFTER
CARDIAC CATHETERIZATION
PROCEDURES**

DIRECTIONS FOR APPLICATION AFTER CARDIAC CATHETERIZATION PROCEDURES

INSTRUCTIONS FOR USE:

1. Open Syvek[®]Patch pouch and deliver it to the sterile field.
2. Hold occlusive pressure proximal to the puncture site and remove the sheath.
3. Allow 1-2 drops of blood from site.
4. Immediately place Syvek[®]Patch over the puncture site and hold firm pressure on the Syvek[®]Patch, while continuing to hold proximal pressure.
5. Slowly release occlusive proximal pressure after 4 to 6 minutes (3 – 4 minutes for diagnostic procedures) while continuing to apply firm compression over the puncture site.
6. Hold firm pressure on the puncture site until hemostasis is achieved, then slowly release.
7. Assess the site without disturbing the Syvek[®]Patch, then cover with a transparent dressing.

Remove Syvek[®]Patch within 24 hours

Soak Syvek[®]Patch with water

Gently peel off Syvek[®]Patch.

STERILE R

EC REP

EMERGO EUROPE
Pleinweg 25
2514 AP The Hague
The Netherlands

CE
0459
15°C 30°C

Marine Polymer
Technologies, Inc.
159 Lorum St.
Tewksbury, MA 01876-1716
1-888-666-2560



Made in the USA

