




EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

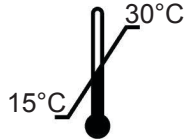

US Patent Numbers

7,041,657 7,115,588 7,931,637  
8,481,512 8,835,408 8,859,528

MPS 500-70 Rev 05 08/2018

# mr<sup>o</sup>dh<sup>®</sup>

A trauma dressing intended for the temporary control of severely bleeding wounds such as surgical wounds (operative, postoperative, donor sites, dermatological, etc.) and traumatic injuries.

**REF** 400-25 (5cm x 5cm)

**REF** 400-14 (10cm x 10cm)

**MPT** MARINE POLYMER  
TECHNOLOGIES

Innovative Medical Solutions



159 Lorum St.  
Tewksbury, MA 01876-1716  
1-888-666-2560

Made in USA

**DESCRIPTION:** mrdh® is a soft, white, sterile, non-woven, lyophilized pad of a cellulosic polymer isolated from microalgae (poly-N-acetyl glucosamine). It is attached to x-ray detectable gauze and packaged in a sterile blister pack.

**INDICATIONS:** mrdh® is intended for use under the direction of a healthcare professional.

mrdh® is a trauma dressing intended for the temporary control of severely bleeding wounds such as surgical wounds (operative, postoperative, donor sites, dermatological, etc.) and traumatic injuries.

**PRECAUTIONS:**

There are no known contraindications to the use of mrdh®.

**mrdh® APPLICATION:**

1. Open sterile blister pack.
2. Place mrdh® on open wound, with active side toward wound surface.
3. Apply firm hand compression until bleeding stops.
4. If the dressing becomes completely soaked through, apply another dressing over the first and continue pressure.

**TO REMOVE:**

1. Soak mrdh® with sterile saline.
2. Gently peel away.

**CAUTION:**

Federal (USA) law restricts this device to sale by or on the order of a physician or other licensed practitioner.

**Not Made with Natural Rubber Latex**

Sterilized with radiation.

Do not resterilize and/or reuse.

Single Use Only.