

DESCRIPTION

SyvekNT® is a soft white, sterile non-woven pad of a cellulosic polymer isolated from microalgae (poly-N-acetylglucosamine), which is attached to medical grade foam backing.

PRECAUTIONS

There are no known contraindications to the use of SyvekNT®. Longer compression may be necessary for hypertensive or obese patients.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician or other licensed practitioner.

Single use only.



EMERGO EUROPE
Prinsengracht 20
2514 AP The Hague
The Netherlands

0459

Marine Polymer
Technologies, Inc.
159 Lorum St.
Tewksbury, MA 01876-1716
1-888-666-2560

INDICATIONS

SyvekNT® is intended for use under the direction of a healthcare professional.

SyvekNT® promotes the rapid control of bleeding in patients on hemodialysis and patients in anticoagulation therapy.

SyvekNT® is indicated for use in the local management of bleeding wounds such as vascular access site, percutaneous catheters or tubes and surgical debridement.



REF 400-09

3 cm × 3 cm

MPT MARINE POLYMER
TECHNOLOGIES

Innovative Medical Solutions

159 Lorum St., Tewksbury, MA 01876-1716
1-888-666-2560

MPS 500-17 rev 12 05/2018
Made in U.S.A.

**DIRECTIONS
FOR APPLICATION AFTER
CARDIAC CATHETERIZATION
PROCEDURES**

DIRECTIONS FOR APPLICATION AFTER CARDIAC CATHETERIZATION PROCEDURES

INSTRUCTIONS FOR USE:

1. Open SyvekNT[®] pouch and deliver it to the sterile field.
2. Hold occlusive pressure proximal to the puncture site and remove the sheath.
3. Allow 1-2 drops of blood from site.
4. Immediately place SyvekNT[®] (Syvek[®] side down) over the puncture site and hold firm pressure on the foam side of the SyvekNT[®], while continuing to hold proximal pressure.
5. Slowly decrease amount of proximal pressure after 4 to 6 minutes (3 – 4 minutes for diagnostic procedures) while continuing to apply firm compression over the puncture site.
6. Hold firm pressure on the puncture site until hemostasis is achieved, then slowly release.
7. Assess the site without disturbing the SyvekNT[®], then cover with a transparent dressing.

REMOVE SyvekNT[®] after 24 Hours:

1. Soak SyvekNT[®] with water.
2. Gently peel off SyvekNT[®].

Not Made with Natural Rubber Latex



Patents:

7,041,657; 7,115,588; 7,931,637; 8,152,750;
8,481,512; 8,834,516; 8,835,408; 8,859,528;
8,992,453